

# FreeStyle Libre 2

FLASH GLUCOSE MONITORING SYSTEM

## Quick Start Guide for FreeStyle Libre 2 Reader



### Setup Overview

Refer to your User's Manual for full System instructions and information.



- 1 Assemble and Apply Sensor to your body
- 2 Start new Sensor with Reader  
Wait 60 minutes for start-up
- 3 Check your glucose whenever you want for up to 14 days

## 1 Assemble and Apply Sensor to the back of your upper arm

### Get Ready

#### STEP 1




Select site on back of upper arm. Do not use other sites as these may result in inaccurate glucose readings.  
**Note:** Avoid scars, moles, stretch marks, lumps, and insulin injection sites. To prevent skin irritation, rotate sites between applications.

#### STEP 2



Wash site using a plain soap, dry, and then clean with alcohol wipe. Allow site to air dry before proceeding.

#### STEP 3

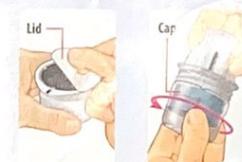

Peel lid completely off Sensor Pack. Unscrew cap from Sensor Applicator.  
**CAUTION:** Sensor codes must match on Sensor Pack and Sensor Applicator or glucose readings will be incorrect.

### Prepare Sensor Applicator

#### STEP 4



Line up dark mark on Sensor Applicator with dark mark on Sensor Pack. On a hard surface, press down firmly on Sensor Applicator until it comes to a stop.

#### STEP 5



Lift Sensor Applicator out of Sensor Pack.

#### STEP 6



Sensor Applicator is ready to apply Sensor.  
**CAUTION:** Sensor Applicator now contains a needle. Do not touch inside Sensor Applicator or put it back into Sensor Pack.


### Apply Sensor

#### STEP 7



Place Sensor Applicator over site and push down firmly to apply Sensor.  
**CAUTION:** Do not push down on Sensor Applicator until placed over prepared site to prevent unintended results or injury.

#### STEP 8

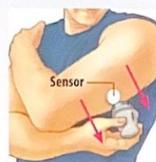

Gently pull Sensor Applicator away from your body.

#### STEP 9



Make sure Sensor is secure. Discard used Sensor Applicator and Sensor Pack according to local regulations.

